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8M91084 (879-9), 8M910985 (879-5), 8M911008 (870-1)

Artikel-Nr.	U <sub>max</sub> / P <sub>max</sub>	Bemerkung
8M910331 (833-1), 8M910332 (832-1), 8M91033 (833-4), 8M91034 (833-5), 8M910912 (802-2), 8M910915 (802-3), 8M910914 (802-4), 8M910915 (802-5), 8M910914 (802-2), 8M910916 (802-5), 8M910934 (802-7), 8M910945 (805-1), 8M910934 (805-5), 8M910934 (805-	1,3 kVp/ 120 W	-
8M910821 (802-8), 8M910822 (802-6), 8M910833	4,3 kVp/	
(805-6), 8M910834 (805-8), 8M910903 (802-9).	120 W	
8M910904 (870-3), 8M910905 (870-9), 8M911008 (870-1).	1,0 kVp/ 120 W	ACHTUNG! Nicht zur Vaporisation sondern ausschließlich zum Schneiden verwenden
8M910808 (879-2), 8M910809 (879-3), 8M910843 (872-4), 8M910906 (879-1), 8M910983 (879-0), 8M910984 (879-9), 8M910985 (879-5).	1,0 kVp/ 180 W	-
Siehe auch Etikett bzw. Katalogangaben.		
Bei der Kombination mit anderem HF-Zubehör entspric der kleinsten Zubehörbemessungsspannung (siehe brauch").		
Die maximale Bemessungsspannung des Produkts ist	dieser Gebrauch	nsanweisung, dem Etikett o

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# Scope

1 Scope

Non-sterlie, reusable product (up to 5x)
Monopolar HF Arthroscopic Electrodes, Article No.: 8M910331 (833-1), 8M910332 (832-1),
8M910333 (833-4), 8M910334 (833-6), 8M910812 (802-3), 8M910821 (802-9), 8M910828 (802-8), 8M910833 (805-8), 8M910834 (802-7), 8M9109134 (802-7), 8M910983 (879-2), 8M910984 (879-8), 8M910984 (879-8)

Article No.	U <sub>max</sub> / P <sub>max</sub>	Note
8M910331 (833-1), 8M910332 (832-1), 8M910333 (833-4), 8M910331 (830-24), 8M9103412 (800-24), 8M910913 (800-21), 8M910914 (800-4), 8M910915 (800-5), 8M910916 (800-2), 8M910916 (800-2), 8M910916 (800-2), 8M9109196 (800-2), 8M910976 (800-2), 8M910979 (800-2), 8M910979 (800-2), 8M910979 (800-2), 8M910979 (800-2), 8M910978 (800-2), 8M91097	1,3 kVp/ 120 W	-
8M910821 (802-8), 8M910822 (802-6), 8M910833 (805-6) 8M910834 (805-8), 8M910903 (802-9).	4,3 kVp/ 120 W	
8M910904 (870-3), 8M910905 (870-9), 8M911008 (870-1)	1,0 kVp/ 120 W	ACAUTION! Not to be used for vaporization, only for cutting.
8M910808 (879-2), 8M910809 (879-3), 8M910843 (872-4) 8M910906 (879-1), 8M910983 (879-0), 8M910984 (879-9) 8M910985 (879-5).		

See also label or catalogue 
When used in combination with other electrosurgical accessories, the maximum rated voltage of the 
combination is determined by the accessory with the lowest rated voltage. (Refer to the "Intended 
Use" section for more details.)
To determine the maximum rated voltage of this product, consult the Instructions for Use (IFU), product label, or the current product catalog.
If you have any doubts or uncertainties ese contact the manufacturer.
If you have any doubts or uncertainties and the product, as well as the IFUs of any accessories 
and the IFE generator being used.

mandatory. This IFU is not intended for users in the USA.

This IFU is not intended for users in the USA.

2 Intended Use

Product intended for application in arthroscopic surgery.

For cutting, coagulating and vaporizing (exceptions see table above) of biological tissue, while applying inguitor fluid see below).

By the used by shilled medical professionals who have been introduced to the product, by using this IFU.

In the be used by stilled medical professionals who have been introduced to the product, eg., using this IFU.

Connect the product to the monopolar/bipolar output of the HF generator via an HF cable, HF handle, or handle with suction.

Activation takes place using a foot switch or an HF handle.

Activation takes place using a foot switch or an HF handle.

Its strictly prohibited to manipulate the electrode tip.

Indication: Product for arthroscopic application for cutting, coagulating and vaporizing biological tissue. The product is intended to be used in large joints, e.g., knee, hip, shoulder, etc.

Supering the product of the

Side effects: Electrolograpa interventions may lead to undesired damages or undesired obusing or unrounding issue.

It is the solor expensive forces during blunt dissection is possible. It is the solor exponsibility of the physician to include this aspect in their professional decision regarding the treatment process selection. All electrosurgical products potentially may lead to muscle stimulation during the application. The product has been designed in such a way that the risk of this undesirable effect is milmized, however, muscle stimulation still can lead to an unexpected movement of the patient in the surgical field.

however, muscle stimulation still can lead to an unexpected movement of the patient in the surgical field.

It cannot be excluded, that the electrode tip wears off earlier than expected when contact with bony surfaces is done intensively.

Connection and activation: Before using monopolar products, ensure that a designated HF naural electrode is correctly applied to the patient and properly connected to the designated HF generator. Selected is correctly applied to the patient and properly connected to the designated HF generator. Selected is correctly applied to the HF panel of the product which the product with the generator. Selected is correctly applied to the HF panel of the Activation is via the buttons on the HF handle or via the foot switch on the HF generator. Combination / Compatibility: Before use, the compatibility of the product with the intended HF handler-HF cable and HF generator has to be verified. In case of uncertainties, contact the manufacturer of this product or the manufacturer of the used accessory of the Generator sells of the accessory (see section "Scope").

The frequency of the used HF generator shall not exceed the maximum frequency of 4 MHz and not accessed the maximum rated voltage of the accessory (see section "Scope").

The strangency of the used with the CSEH of the product of the section of the comparable combination product. Article No. 800-0 to 800-10, 801-0 to 801-10, 832-1, 832-5, 833-1 to 833-9 are only to be used with REGER for Evale Article No. 80208 to 92080, 92815:

Feasible HF generators for the aforementioned monopolar articles:

ERBE KLS Martin EMED BOWA EMED AFTTO

ERBE	KLS Martin	EMED	BOWA	EMED ARTro	]
Covidien	ValleyLab	Tekno	Berchtold	EMED ARTro distrib- uted by REGER	
Feasible HF-gen	erators for aforem	entioned bipola	ar articles:		
ERBE VIO	KLS Martin	EMED	BOWA	EMED ARTro	1
Depuy Mitek VAPR	Artthrocare	Tekno	Berchtold	EMED ARTro distrib- uted by REGER	1

UAPP. Use in combination with compatible products and connectors. The instructions, safety and warning notes in the instructions for use of the accessories and the HF Follow instructions and warnings in the IFU of HF cable and HF generator. It is recommended to use a smoke evacuation system.

MPORTANT:
Handle with utmost care.
This does not only apply for the duration of the surgery but also for the complete duration of storage, processing, and transport as well as during the process of connecting the product with the respective HF handle, HF cable, and/or HF generator.
This applies especially to the thin tips and other sensitive areas, e.g., the insulation. Improper use will result in immediate loss of warranty.
No liability is accepted for any damage that may occur.

3 Safety Notices — WARNING!

Make sure the User Manual of the HF generator has been read carefully before using the product.

Make sure the User Manual of the HF generator has been read carefully before using the product. Proceed according to the maximum rated voltage stated in this IFU or label. Before each use, the product must be cleaned, disinfected, and sterilized according to a validated procedure (DIN EN ISO 17665). (See section "Reprocessing: Cleaning, disinfection, and steriliza-tion").

sun').
In case of uncertainties, contact the manufacturer.
Before usage, a visual inspection and function test has to be done (see section "Visual Inspection Test").

runction | lest').

Before usage, a visual inspection has to be done for pressure points of damage on the product in case of damage, deformation, or similar is detected on the product and its insulation, the or most not be used.

In Case or damage, decimands, and must not be used.

It has to be replaced with a new one. 
Irrigation fluid for monopolar products:

During use, the active electrode must always be completely surrounded by 100% electrolyte-free 
firshing solution, e.g., use Purisol. Do not use without rinsing solution. 
Irrigation fluid for bipolar products:

When in use, both the active electrode and the neutral electrode of the product must always be completely surrounded by 100% conductive flushing liquid.

Les 0.9% salient solution or Ringer's solution, for example.

NEVER use non-conductive irrigation fluid (e.g., sterile water, Glycine, Purisole, etc.) for bipolar prodicts.

ucts.

Ensure sufficient and adequate flow of irrigation fluid to the joint (at least 100 ml/min.)

At least one (1) sterile backup product has to be available for the application.

It is the responsibility of the user to determine the appropriate product size according to the sional judgment and based on the patient's specific indication, preferred surgical technique.

sional judgment and based on the patient's specific indication, preferred surgical technique, history, etc.

Before application, verify that the product is inserted firmly in the HF handle / HF cable.

This must be done carefully, to avoid damages to the product and/or injuries to the patient, surgical personnel, or third party.

Excessive force can damage the product.

Therefore, the product has to be observed during the complete application.

Exclusion:

Do not activate the product as long as it is in contact with metal objects and/or optics.

During an electrosurgical intervention, the patient must not come into contact with grounded metal objects such as surgical desk frames, instrument trays, etc.

When the contact with the description of the product and the product and the product and the product of the product of

plosion.

Do not touch the product tip as long as it is activated.

Do not touch the product tip as long as it is activated.

After switching off the electrosurgical current, the product tip still may be hot, so this could le undestrable burns undestrable burns.

While the product is activated, do not insent or witdraw the tip from the surgical site. Inadvertent activation of the product or moving the tip outside of the field of vision can lead to

tentional injury of tissue.

During activation, make sure the product does not contact the arthroscope as thereby the product and/or arthroscope may be damaged.

Do not activate the product uniterruptedly over a longer period.

Do not use metal cannulas as they may damage the product insulation or create an alternative current path (capacitive coupling), resulting in uniterational burns.

It is recommended to use plastic cannula systems.

It is the responsibility of the user to select the lowest possible power setting on the HF generator to achieve the desired effect during the respective operation.

During the surgical procedure, mechanical forces may cause deformation or wear of the product. If alreads not the product is defected during the application, the product has to be exchanged for a desired product may be reused.

Only an underaged product may be reused.

one. siston, adhesion of tissue, discoloration, sooting, etc. do not represent a reason for complaint and Â

not permit claiming the manufacturer's warranty. (Re) Processing: Cleaning, disinfection and sterilization

do not permit claiming the manufacturer's warranty.

4. (Re) Processing: Cleaning, disinfection and sterilization

4.1 Maximum number of reprocessing cycles

Due to the materials used and due to the intended use of the products, the articles listed in the Scope of this IFU only are allowed to run through a maximum of up to 5 reprocessing cycles (refer to section Visual Inspection and Function Test'). When applied according to the Intended Use, the product underlies natural wear and tear, considering manner and draution of the application as well as manner and frequency of reprocessing. Therefore, a visual inspection and function test has to be done before each usage. (Refer to section Visual inspection and function test, in particular the condition of the insulation and the product tip. Visual inspection and function test, in particular the condition of the insulation and the product tip. 42. Time requirements for cleaning and automated cleaning and disinfection has to be done immediately after the application. Newer on talter than 1 hour after the application of the product, deposit the product carefully (protection of lifetime of the product). After application of the product, deposit the product carefully (protection of lifetime of the product). After application, separate contaminated product and deposit in a suitable containiner (doposit neems 'do not drop'). Immediately after the application, the magnetic ordinarine (doposit neems 'do not drop'). Immediately after the application of the product, deposit the product and deposit in a suitable containiner of the product and deposit in a suitable containiner deposit neems 'do not drop'). Immediately after the application of the product and deposit in a suitable containiner deposit neems 'do not drop'). Immediately after the application of the product and deposit the way that contamination of the transport and an and any transport containiners are cleaned and disinfected after transport as well.

4.4 Validation of (Re)Processing

Brochure, pages 30-32). Take care, all and any transport containers are cleaned and disinfected after transport as well.

4.4 Validation of (Re)Processing procedure is recommended. Equivalent deviant processes are possible. The following validated processing procedure is recommended. Equivalent deviant processes are possible. Then, it is the sole responsibility of the user to safeguard the suitability of the applied procedure by suitable means (e.g., validation, routine monitoring, verification of material compatibility, etc.). The following processing requirements of the following processing requirements specific to the respective clinical place (operator) as well as national or country-specific regulations have to be followed as well. Never use sharp objects for cleaning. Disinfectants always have to be fined and removed carefully.

4.5 Preparation for Cleaning. Place it in a container provided for cleaning. Place it in a container provided for cleaning. It is not necessary to disassemble the product. It is not necessary to disassemble the product. Pre-clean the product immediately. At the latest, however, 1 h after completion of the application. Uses tap water (potable water quality) (<40°C) and aldehyde-free, non-fixing disinfectants if applicable.

plicable.

Thoroughly remove the surface stain with a soft brush or synthetic fleece, as otherwise particles or dried secretions may adhere.

Ensure that hard-to-reach areas are well cleaned and rinsed several times. This may make subsequent cleaning and sterilisation difficult or impossible.

Cartiles and Luman have to be rinsed intensively using at least 3x20 m toold tap water (<40°C) with he add of a rinsing addeter (e.g., from the company Medisafe), or with a syringe or with a water jet Cavities and lumen have to be make municipally and the aid of a mining adapter (e.g., from the company Medisale), or with a synnge or white a pistol (>30 Sec.).

This pre-cleaning step always has to be done prior to the manual cleaning or cleaning with the cleaning and disinfection device (CDD).

47. Manual Cleaning and Disinfection

Prepare an immersion bath with a suitable fluid cleaning agent.

Use a cleaning agent that is compatible with the disinfectant and suitable for immersion baths.

Follow the instructions of the manufacturer of the cleaning agent and disinfectant.

Only use agents suitable for medical devices made from metal and plastics with a pH value between

Only use agents suitable for medical devices made from metal and plastics with a pH value between 5 and 12.3. Recommendation: Cleaning agent gigazyme@ (Schülke & Mayr) and disinfectant Korsolex Plus. Onct use high alkaline cleaning agents. The service of the product of the product product of the product of the product Prepare a separate immersion bath with disinfectant according to the specific cleaning agent IFU. Prepare a separate immersion bath with disinfectant according to the specific disinfectant IFU. Immerse the product completely in the ultrasonic bath with a cleaning agent (e.g., 0.5% gi-gazyme@)

Immerse the product completely in the ultrasonic bath with a cleaning agent (e.g., 0.5% gi-gazyme®).
Clean product in an ultrasonic bath using a sonication time of 5 Min. and a frequency of 45 kHz.
Follow all instructions outlined in the IFU of the cleaning agent, disinfectant, and ultrasonic bath.
Ensure the product will not touch other products or parts in the ultrasonic bath.
Ensure sonic shadows in the ultrasonic bath are avoided.
Then, clean the product with a soft brush under cold running town water (<40°C).
Intensively rinse cavities and lumen with a water jet pixtl of 30°S coj or similar for at least 1 Min.
Afterwards, rinse the product throughly for at least 1 min. under running tap water (>40°C) to remove any residues of the cleaning agent.
Inspect the product visually on the remaining stain.
In case stain is still present, repeat the aforementioned cleaning steps as long as it needs until no soil is present.

In case stain is still present, repeat the aforementioned cleaning steps as long as it needs until no soil is present.

Afterwards: Immerse product completely in a disinfectant bath including e.g., Korsolex Plus 3%, for at least 15 Min.

Follow the manufacturer's data for residence time.

Ensure the disinfectant will contact all areas on the product.

Ensure the disinfectant will contact all areas on the product.

Alterwards: Rinse the product thoroughly for at least 1 Min. with demineralized cold water, to remove all disinfectant residues.

Additionally: Rinse all narrow and areas difficult to access, all cavities and lumen with a syringe several times (at least 3), using each time 20 mil cold demineralized water.

Dry product with a lint-free wipe and sterile compressed air.

4.8. Automated Cleaning and Disinfection

Only use cleaning and disinfection devices (CDD) with proven efficiency according to EN ISO 15883. Follow the data of the manufacturer of the cleaning and disinfectant machine.

Only use agents suitable for medical devices small form metal and plastics with a pH value between 5.5 and 12.3. Apply program to the cleaning and disinfectant machine.

5.5 and 12.3.

Recommendation: neodisher® medicilean forte (Dr. Weigert GmbH & Co. KG).

Apply program for thermal disinfection.

Follow instructions and data regarding program courses and machines.

Product has to be stored safely and protected against mechanical damages during automated cleaning and disinfection.

Do not clean together with sharp-edged or pointed objects.

Follow data for loading of the cleaning and disinfection device (CDD).

Use rinsing adapters for products with the lumen and connect them according to the instructions in the User Manual of the cleaning and disinfection device (CDD).

Cleaning Program

Start the program course with the following parameters:

1 Min, per-finsing with cold water

Emplying

3 Min, pre-finsing with cold water

Emptying 3 Min. pre-rinsing with cold water

Emptying 5 Min. cleaning at 55°C with 0.5% alkaline cleaning agent

Emptying 3 Min. neutralization with warm tap water (>40°C) and neutralizer (0.1% Neodisher® Z)

Emptying
 Emptying
 Disinfection Program
 Automated thermal disinfection considering national requirements regarding AO value (see EN ISO 15883, AO value >3000).
 5 Min. cleaning at 92°C +/-2°C
 Coul

Drying
30 Min. at 90°C

30 Min. at 90°C.

Remove firshing adapter
the end of the program course, remove the product and inspect it on the remaining stain.

The standard of the program course, remove the product and inspect it on the remaining stain.

The standard is not standard to the standard s

Les proviets in a single-use sterilization packing (double packing) from paper of foli or put the product in a stalification container.

Respect requirements for sterilization packaging according to EN ISO 11607 and EN 868.

49. Sterilization
Only products that have been cleaned and disinfected are allowed to be sterilized.
Only apply steam sterilization in an autoclave (fractioned pre-vacuum with sufficient product drying) for this product.
Adjust sterilization parameters:

Minimum 1347 can display the steam of maximum 137°C in saturated steam.

Hoding time at least 5 Min. until max. 20 Min.

Example: Sterilizer Class B, manufacturer Tuttnauer.
Respect data of the Sterilizer amundacturer regarding the load, handling, and drying times.

Exclusion:
Do not apply bot air, EO-gas, Radiation or Plasmo for sterilization. sion: apply hot air, EO-gas, Radiation, or Plasma for sterilization, or any other sterilization

ORTANT: Before usage, let the product cool to room temperature.
the sole responsibility of the user to maintain the sterile condition of the product after

Zation process.

In case the aforementationed chemicals and machines for cleaning, sisferction, or startilization are not have been aforementationed thermicals and machines for cleaning, sisferction, or startilization are not have been accordingly of the user to validate the procedure actually applied.

Also, if a startilization mention do there than that described above is applied, this deviating procedure has to be validated by the user accordingly.

4.10. Limitation of Reprocessing

Product life depends on wear and tear, handling, application time, damage and frequency and type of reprocessing.

Before usage, the complete product, especially insulation and product tip have to be inspected for pressure points and damages.

Products exhibiting damage or pressure points are not allowed to be used and have to be replaced

nly to be used by skilled medical professionals who have been introduced to the product. addition to the acknowledged benefits of HF technology, the application includes several risks that

niques. The HF generator, which is used in combination with the product, has to be checked for damages Follow the instructions outlined in the user manual of the manufacturer.

etc.) in the immediate vicinity during the entire application, otherwise there is a risk of explosion.

Ensure correct patient positioning, i.e., use insulating OP drapes that are liquid-tight, dry, and absor

Ensure correct patient positioning, i.e., use insulating OP drapes that are liquid-tight, dry, and ab bent.

Insulate conductive areas and contact points from the patient.

Place dry cellulose tissues in skin folds, breast folds, and between the extremities.

Any fluid that possibly pools in body cavities has to be removed before the procedure.

Only use non-flammable disinfectants.

Do not use alcoho-based functures, for example.

Remove all body jewelry from the patient.

Putting a band-aid over the body jewelry is not sufficient!

All oxygen connections must be light and leak-proof during the procedure.

As long as the product is not applied, place it on a dry, clean and non-conductive and well-vis surface, that is not in contact with the patient.

Never place the product on the patient.

Unitended activation of the product can lead to unintentional burns or other injuries to the patit the user, or a third party.

Never wrap cabbies around the patient and never lay cables over the patient.

Operation:

Insert the product tip through the arthroscopic portal into the surgical field of vision. Ensure the till trest the product tip through the arthroscopic portal into the surgical field of vision. Ensure the till trest the product tip through the arthroscopic portal into the surgical field of vision. Ensure the till

peration:
int the product tip through the arthroscopic portal into the surgical field of vision. Ensure the tip of a product is fully submerged in a conductive fluid.
It sativate the product in the immediate vicinity of the tissue to be treated, WARNING: an active viduct tip mast NEYER be outside of the user's field of vision during the surgery as this can cause

Do not buch the tip of the instrument during the entire use 
No continuous activation.

Apply only short activation times.

Apply only short activation times.

At least up to 6 minutes cumulated ablation time and no longer than up to 6 seconds single activation.

Keep on longer breaks between activation phases.

Only adjust low power settings.

All electrosurgical instruments can potentially cause muscle stimulation during use.

The design of this product has been chosen to minimize the risk of his unwanted effect.

Nevertheads, suscle stimulation may cause unexpected patient movement in the surgical field.

End of surgery.

At the end of the surgery, remove the product carefully from the patient's body and confirm the completeness of the system.

Switch off the HF generator.

Switch off the HF generator.

REGER Medizionethonik GmbH

9 Mra REGER Medizi

REGER Medizintechnik GmbH Gewerbestrasse 10 | 78667 Villingendorf | Germany. Tel: +49 (0) 741 270 698-0 | Fax: +49 (0) 741 270 698-10

med.de | www.reger-med.de Swiss Authorized Representative

Weissensteinstr. 26 | Ziegelmattareal Bau 6 | 4503 Solothum | Switzerland Tel: +41 (0)32 621 60 01 | Fax: +41 (0) 32 621 60 02

11 Warranty
Our products satisfy the highest quality standards. Liability and warranty are excluded for all products

Our products satisfy the highest quality standards. Liability and warrany are excluded for all product that have been modified in any way, not applied according to their intended Use, that has been han-dled or applied improperly, or in case of any other deviation from the instructions outlined in this IFU. Furthermore, REGER Medizintechnik GmbH denies any liability for any accidental, intentional dam-age or damage or loss arising out of handling or application of the product. Additionally, all liability and warranty are omitted in case our product was repaired by a company that has not been authorized by us. Unauthorized repairs are strictly prohibited.

Return Returned products will be accepted only if they are marked as "hygienically safe" or nated" and have been securely packaged for shipping.
Use our Return Form for returns.

medical devices have to be followed as well.

14 Regulatory Remark

Due to regulatory reasons, we advise users and patients that in ection with our medical device must be reported to the manufact in which the user and / or patient is resident.

15 About these instructions for Use

he instructions for use mus eriod of use of the product For a current revision of these IFU, please contact medimex GmbH or visit the medimex Homepage for a download of this document:

Products exhibiting damage or pressure points are not allowed to be used and have to be replaced with new ones.

During and after application, tissue may adhere to the product, or sooting may be present on the distal end of the active electrode.

Such adhesions or sooting do not represent a reason for complaint and the product has to be exceeded to the product of the product product has to be exceeded to the product product has to be exceeded to the product has to be replaced with a new one.

Blockage of the suction channel (if applicable), does not represent a reason for complaint. A product, that exhibits a blocked suction channel (if applicable), has to be replaced with a new one. Before usage, an electrical continuity test has to be done.

In case the product does not pass the electrical continuity test, it is not allowed to use the product any longer and has to be replaced with a new one. If damaged, do not use these products.

Never lay this product on the patient or in their direct vicinity.

Exclusion of Repair and Modification

Defective products are not allowed to be repaired.

Exclusion of Repair and Modification

Defective products are not allowed to be repaired.

Insultrative modification or repair is strictly prohibited and leads to an immediate loss of the manufacture modification or repair is strictly prohibited and leads to an immediate loss of the manufacture of the modification or repair is strictly prohibited and leads to an immediate loss of the manufacture of the modification of the manufacture of the modification of the manufacture of the modification of the mod

7 Packaging, Storage, Transport
The product must be stored in a clean and dry environment, they must be protected from direct sun-light.
Always handle the nording with the control of the product with the control of the pro

light. Always handle the product with the utmost care when transporting, cleaning, disinfecting, maintaining, steritizing, and storing it.

This is especially true for fine tips and other sensitive areas (e.g., insulation). Do not store or transport the product together with sharp-edged or pointed objects.

Store only in protective containers with individual compartments or individually shrink-wrapped. The maintenance of the sterile state after the sterilization process must be ensured by the operator.

8 Safety, installation and application

Only to be used to skilled medical or professionals who have been introduced to the product.

If adulting the acknowledged between or in rectifications, the appreciant includes several issistant mayor use and failure to follow the instructions for use may result in unintentional burns to the adulting use of the properties of the surgical personnel is recommended. In the operator responsibility to ensure the training of users/surgeons in arthroscopic surgical tech

Follow the instructions outlined in the user mature in the international Preparation:
Preparate the HF generator according to the User Manual and place it for application. Proceed with the generator according to the User Manual.
Bring the sterilized product into the sterile field and confirm the integrity of the product's insulation by visual inspection.
Confirm the integrity of the insulation of the product by visual inspection.

Confirm the integrity of the insulation of the product by visual inspection.

At the beginning of the application, make sure that the cable of the handle is connected properly to
the HF generator and that the correct settings are selected and displayed.
It is the responsibility of the user to select the lowest possible power setting on the HF generator to
achieve the desired effect during the respective operation.

Note: when using products with suction, connect the suction adapter to a tube that leads to a suitable
suction system (e.g., suction canister, wall suction system, etc.). The minimum vacuum target value
is 300 mmHg.

The instructions for the correct use of the HF neutral electrode, including patient protection and mon-

toring, monitoring of the HF neutral electrode and all other instructions, safety instructions and warnings in the instructions for use of the HF neutral electrode must be observed. Make sure that there are no flammable substances (anesthetics, oxidizing gases, endogenous gases

product by must NEVER be outside of the user's field of vision during the surgery as this can cause unintentional fissue damage. Whenever the product is not in use, place it on a dry, clean, non-conductive, and highly visible field that is not in contact with the patient. Accidental activation of the product may cause burns to the patient. Do not touch the tip of the instrument during the entire use No continuous activation.

www.medimex.de/downloads
Changes reserved.
This IFU is not intended for users in the USA.

©2025 REGER Medizinethnik GmbH All rights reserved. All company names, service marks, trademarks, product names and logos appearing on this document are the property of their respective owners and shall be treated ac-

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Alle Rechte vorbehalten. Technische Änderungen vorbehalten All rights reserved. Subject to technical chances without further notice