# medimex Reinigungsindikatoren H

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# medimex Cleaning indicators H

## Instruction manual in English

### Indicators for routine checks in washer-disinfectors disinfection devices (RDG) according to DIN EN ISO 15883-2

#### Preliminary remarks

In the cleaning process, four factors (SINNER circle) determine the result:

1. the time,
2. the mechanics,
3. the temperature and
4. the chemistry

Factors 1 to 3 are specified by the washer-disinfector and matched to the cleaning agent.

Faults in the process, especially the effectiveness of the cleaning agent, can be detected by using sufficiently critical cleaning indicators such as the medimex Cleaning indicators.

The medimex Cleaning indicators are characterized by high adhesion and can only be removed with alkaline cleaning agents, which are typically used, if the corresponding exposure time and the pressure of the spray nozzles are observed.

The use of medimex Cleaning indicators does not exempt the user from checking every cleaning object for cleanliness in accordance with the guidelines of the RKI.

#### On the use of medimex Cleaning indicators H

The objectives of reprocessing medical devices in washer-disinfectors (WD) in accordance with DIN EN ISO 15883-2 are:

- the reliable removal of typical soiling
- Thermal disinfection with demineralized water
- Drying of the cleaned and disinfected products.

DIN EN ISO 15883-5: "Performance requirements and criteria for test methods for the verification of cleaning efficacy" is intended exclusively for type testing and performance qualification testing of washer-disinfectors. The test soiling contained therein is unsuitable for daily routine testing.

medimex Cleaning indicators for routine testing must reveal deviations in key process factors that are not registered by internal device monitoring. These include, among other things, the loading pattern, detergent quality and nozzle permeability.

medimex Cleaning indicators must be storable, easy to use and clear in their assessment.

#### Application of the medimex Cleaning indicators H

The medimex Cleaning indicators are inserted into the appropriate holder and placed at a critical point on the screen basket for cleaning or adapted to a spray nozzle. Functional tests should be carried out before use in routine inspection. After the cleaning process, the indicator must be free of the red test contamination.

| Appearance of the indicator                     | Possible causes                                    | Activities                                                                                                         |
|-------------------------------------------------|----------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|
| No more red test soiling present                | Process flow OK                                    | None                                                                                                               |
| The red test soiling is still completely intact | No or unsuitable cleaning agent                    | Check the manufacturer's instructions for the cleaning agent                                                       |
| Remains of the red test soil are visible        | Insufficient mechanical action on the test soiling | Checking the loading with regard to barriers in front of the indicator, insufficient pressure of the spray nozzles |

Atypical soiling, such as adhesive residue from adhesive plasters or blood encrustations in joints, e.g. on needle holders due to excessive standing times, must be cleaned manually or in an ultrasonic bath in a targeted manner.

#### Delivery information

The medimex Cleaning indicators are available in the following versions:

##### medimex Cleaning indicators H

Size: 10 x 40 mm

##### medimex Cleaning indicators S

Size: 5 x 60 mm

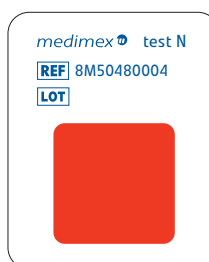
Front and back

##### medimex Cleaning indicators N

Size: 28 x 34 mm

A holder for the medimex Cleaning indicators N indicators is included in every pack.

##### medimex Cleaning indicators L

Size: 25 x 90 mm

The sizes of medimex Cleaning indicators offered can be used in various commercially available holders.

#### Storage conditions Environmental protection

After each removal, the product must be stored in a closed foil bag at room temperature and a relative humidity of 30 to 60 % to prevent undesirable environmental influences.

All medimex Cleaning indicators contain no toxic or water-polluting substances.

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medimex GmbH