medimex Reinigungsindikatoren U

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medimex cleaning indicators U

Instruction manual in English

Indicator for routine testing of the effectiveness of ultrasonic cleaning devices

Preliminary remarks on ultrasonic cleaning

Ultrasonic cleaning is a very effective method for removing dirt from hard-to-reach areas. Ultrasonic cleaning devices are used in the reprocessing of medical devices during manual cleaning or for pre-cleaning. This effectively removes blood and tissue residues from the joints of instruments and hollow bodies. However, they are not part of the DIN EN ISO 15883 series of standards. The principle of ultrasonic cleaning technology is based on cavitation. In this process, a cleaning fluid is stimulated by ultrasonic waves so that small bubbles, so-called cavitation bubbles, are formed. The bubbles implode, thus removing the deposited dirt particles without the need for abrasive chemicals. Cavitation is therefore the mechanical factor in ultrasonic cleaning. For greater effectiveness, suitable cleaning agents should be used.

Notes on the use of medimex cleaning indicators U

Cleaning indicators have the task of reproducibly testing the effectiveness of all cleaning factors: time, mechanics, temperature and cleaning agent. The specially developed medimex cleaning indicators U with the corresponding test device (article no. 8M50480006), which simulates a difficult-to-access instrument, meet these requirements.

The foil test recommended in the Guidelines for the Validation of Manual Cleaning and Manual Chemical Disinfection of Medical Devices (IEC/TR 60886:1987-03) is intended for functional testing only (see recommendation of the Quality Committee).

Application notes for the medimex cleaning indicators U

The medimex U cleaning indicator is inserted into the slot of the test specimen spigot. Ensure that the green test soil points towards the irrigation channel, see picture above. The test specimen is screwed together and placed in the basket with the instruments, see picture below.

Evaluation of the medimex cleaning indicators U

The medimex U cleaning indicator is inserted into the slot of the test specimen spigot. Ensure that the green test soil is facing the irrigation channel, see image below left. The test specimen is screwed together and placed in the basket with the instruments, bottom right image.

Appearance of the indicator	Possible reasons	Action
No green test soiling remains present	Process flow OK	None
The green test soiling is still completely intact	No or insufficient mechanical impact from ultrasound	Testing of the devices functionality
Remains of the green test soil	Insufficient dirt removal by the cleaning agent	Checking the cleaning parameters

Storage conditions for environmental protection

medimex U cleaning indicators must be stored in a sealed foil pouch after each use at room temperature and a relative humidity of 30 to 60%. All medimex U cleaning indicators do not contain any toxic or water-polluting substances.

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medimex GmbH