Electrosurgical Standard Smoke Evacuation Handpiece Instructions for Use

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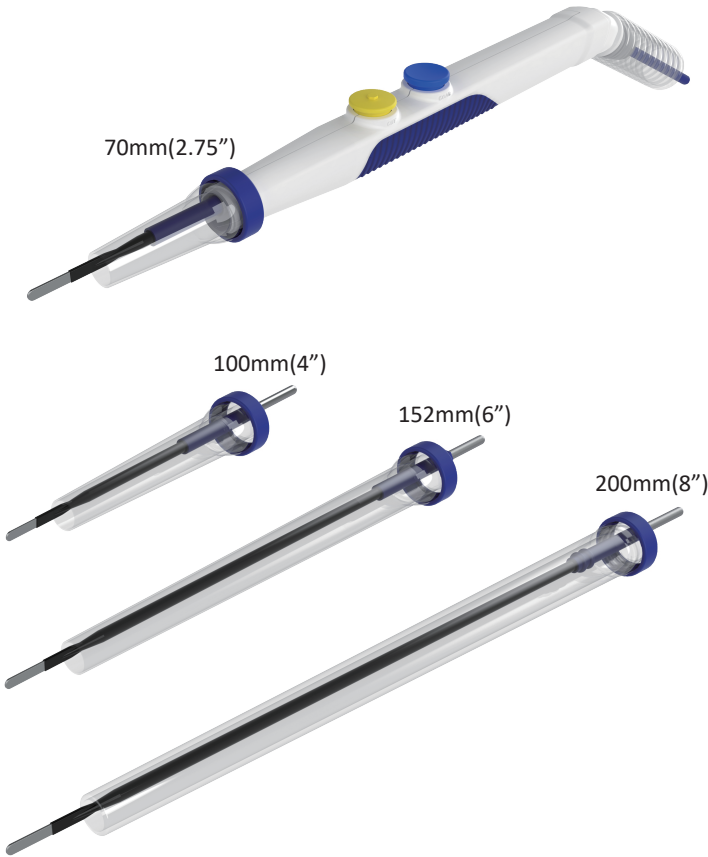
MEDSTAR
TECHNOLOGY
Zhejiang Medstar Technology Co., Ltd.
1/F, Bldg.1, No.239 Yatai Rd.,
Nanhu District, Jiaxing, 314050, China

EC REP

OBELIS S.A.
BOULEVARD GENERAL WAHIS 53
1030 BRUSSELS, BELGIUM

EN

Description/Intended Use:

- Surgical smoke has been shown to contain a large number potential of Biological and Chemical hazards. It also creates an offensive odour and reduces visibility for the surgical team.
- The plume contains Carcinogens, Poisons, Infective Virus and Bacteria.
- The Medstar standard smoke evacuation pencil is designed to be used in conjunction with 70mm/100mm/120mm/152mm standard electrodes and Medstar integrated smoke evacuation leap electrodes .
- The Medstar standard smoke evacuation pencil, when used with an effective smoke evacuation system, removes smoke plume from the surgical site. The pencil enables the operator to remotely conduct an electrosurgical current from the output connector of an electrourgical unit to the operative site for the desired surgical effect.
- Maximum voltage is not to exceed 6 kV peak.

Contraindications:

These devices should never be used when:

- There is visible evidence of damage to the exterior of the device such as cracked or damaged plastic or connector damage.
- These devices fail the inspection described herein.
- In the presence of flammable gases, flammable prep solutions or drapes, oxidizing gases such as Nitrous Oxide(N2O), or in oxygen-enriched environments.
- Do not use monopolar electrosurgery on small appendages, as in circumcision or finger surgery.

Electrosurgery is potentially hazardous for patients with pacemakers, AICDs, neuro-stimulators or other active implants. High frequency energy may damage or adversely affect active implants. Ventricular fibrillation may occur.

Warning:

- DANGER!! The smoke evacuation pencils are not intended to suck fluids from the surgical site. Suction of fluids may cause electrosurgical unit short circuit and “CUT”&“COAG” button AUTOMATIC ACTIVATION.
- Keep active accessory away from the patient when not in use. An Accessory Holster may be used to hold active accessories safely.
- Do not use in the presence of combustible or explosive materials!
- The ability to effectively clean and re-sterilize this single patient use device has not been established and subsequent re-use may adversely affect the performance, safety and/or sterility of the device.

Caution:

- The use-by-date is indicated on the packaging. Do not use this product if the use-by-date has been exceeded!
- Make sure to remove tip protector before using.
- Use lowest possible power setting on the associated electrosurgical unit capable of achieving desired surgical effect.
- Activation time should be as short as possible.
- Never allow the associated accessories connected to these devices to be in contact with skin of the patient or operator except during intended activations.
- Do not permit the cables connected to the associated electrosurgical accessories to be parallel and in close proximity to the leads of other electrical devices.
- Always place unused associated electrosurgical accessories in a safe insulated location such as a holster when not in use.
- Inspect and test each device before each use.
- After use, this product may be a potential biohazard. Handle and dispose of as required by hospital policy and applicable law.

Inspection :

These devices should be inspected before each use. Visually examine the devices for obvious physical damage including:

- Cracked, broken or otherwise distorted plastic parts.
- Broken or significantly bent connector contacts.
- Damage including cuts, punctures , nicks, abrasions, unusual lumps, significant discoloration.
- Verify that the electrode is fully and securely seated in the handpiece before use.

Instructions for Use:

- Remove the tip protector from the handpiece prior to use.
- Ensure that the electrode is firmly seated in the handle.
- Ensure that the smoke cover is firmly seated in the handle.
- If you change the electrode, make sure that it is fully inserted into the handle.
- Pull the securing collar off the cable.
- Plug the 3-prong plug into the appropriate receptacle on the electrosurgical unit. Press the yellow button on the handpiece to activate cutting mode. Press the blue button on the handpiece to activate coagulation mode. Make sure that the electrosurgical unit responds correctly.
- Connect tubing adapters to the smoke evacuator.
- NOTE: Standard Adapter is 22mm female, 22mm male adapter is also available, please consult Sales Representative.
- Change of electrode+smoke cover
 - 1) twist the smoke cover in anticlockwise direction and remove the smoke cover
 - 2) pull out old electrode
 - 3) insert new electrode fully into the handle
 - 4) select correct smoke cover and twist in clockwise direction, make sure that the cover is firmly seated on the handle.
- NOTE: smoke cover is X-Ray detectable, available for electrodes length of 70mm(2.75") 100mm(4") 120mm(4.75") 150mm(6") 200mm(8")

Symbols used on labeling:

Catalogue number	Batch Code	Use-by date	Date of manufacture
Do not re-use	Dose Not Contain or presence of natural rubber latex	Keep away from sunlight	Keep dry
Consult instructions for use	Caution	Do not use if package is damaged	Manufacturer
Humidity limitation	Temperature limit	Sterilized using ethylene oxide	
Product complies with requirements of DIRECTIVE 93/42/EEC MEDICAL DEVICES	Authorized representative in the European Community		

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